

N95 Respirator Mask collection instructions

Battelle in partnership with Cardinal Health

Due to the critical shortage of hospital supplies, Battelle and Cardinal Health are collaborating to provide an end-to-end solution for collecting, decontaminating and returning N95 Respirator Masks to healthcare providers. As a Battelle preferred partner, Cardinal Health will utilize its Sustainable Technologies™ business for this initiative, a leading provider of single-use device collection, reprocessing and recycling services in the U.S.

The Battelle CCDS Critical Care Decontamination System™ is authorized for use by the U.S. Food and Drug Administration (FDA) under an Emergency Use Authorization (EUA). Cardinal Health Sustainable Technologies™ will expedite the logistics of collecting used N95 Respirator Masks from a customer facility, transporting such respirators to a designated Battelle CCDS™ site and returning decontaminated N95 Respirator Masks back to the originating customer facility.

Healthcare professionals must adhere to the instructions below for proper marking and collection procedures.

Preparation

Label individual N95 Respirator Mask using permanent marker with the following:

1. Three-letter customer site code
2. Two-digit/letter customer site location identifier

Collection

Place N95 Respirator Mask in collection bin provided by Cardinal Health Sustainable Technologies™ in the designated collections area.

____ _ - ____ _
Site code Site location ID

Important: This collection bin is for N95 Respirator Masks only. Do not place any other Personal Protective Equipment (PPE) in this collection bin.

