

# Conserve PPE-Reuse Guide

After gown and glove removal...



## Setup

1 Sanitize hands.

2 Put on fresh gloves.

3 Place wipe on table.

## N95

4 Remove eyewear and place on wipe.

5 Sanitize gloves.

6 Remove N95, lower strap and then upper. Store mask in container or bag. If container used, disinfect container between use.

7 Sanitize gloves.

## Face Shield

8 Wipe front and back of shield.

9 Wipe elastic band.

10 Wipe foam band.

11 Wipe table.

12 Place shield upside down to dry.

13 Sanitize gloves.

14 Remove gloves.

15 Wash hands with soap and water.

## Goggles

8 Wipe front and back of lens.

9 Wipe both ear pieces.

10 Wipe table.

11 Place goggles on clean table to dry.

12 Sanitize gloves.

13 Remove gloves.

14 Wash hands with soap and water.



Questions? Contact your Infection Prevention Department

DISCLAIMER: COVID-19 information is rapidly changing and documents will be updated accordingly.

Distributed by: Patient Safety National Office  
March 23, 2020